

Medical compression garments from Juzo guarantee therapeutic graduated compression. The pressure decreases continuously from the ankle or wrist section upward towards the distal end. We can provide a wide variety of garments, from below-knee stockings to pantyhose, from gloves to face mask, depending on the therapeutic application.

RTW - Standard / MTM - Custom

Mild Compression	RTW - Circular	15-20 mmHg		
			<ul style="list-style-type: none"> · minor obstructive disorders accompanied by feelings of tiredness or heaviness in the legs · isolated varicose vein formation · preventative measure during pregnancy · therapeutic treatment of scars 	
Moderate Compression	RTW - Circular	20-30 mmHg	MTM - Flat / Circular Knit	18-21 mmHg
				<ul style="list-style-type: none"> · varicose veins · leg swelling during pregnancy · after varicose vein sclerosis or operations · for the long-term treatment of healed leg ulcers · propensity for edema formation exists · therapeutic treatment of scars
Firm Compression	RTW - Circular	30-40 mmHg	MTM - Flat / Circular Knit	23-32 mmHg
				<ul style="list-style-type: none"> · varicose veins accompanied by damage to the deep leg veins · after healing of venous ulcers · when a severe swelling tendency exists · chronic venous insufficiency · severe propensity for edema formation
Strong Compression	RTW - Circular	40-50 mmHg	MTM - Flat / Circular Knit	34-46 mmHg
				<ul style="list-style-type: none"> · severe lymphedema · severe chronic venous insufficiency · management of active venous ulcers
Extra Strong Compression	Circular	N/A	MTM - Flat Knit	50+ mmHg
				<ul style="list-style-type: none"> · severe post thrombotic conditions · severe lymphedema · elephantiasis

Prescription Recommendation for Venous Disease

Indications	Juzo Naturally Sheer	Juzo Power Collection	Juzo Basic	Juzo Soft	Juzo Dynamic	Juzo Move	Juzo Silver	Juzo Compression Wrap
Minor varices								
Beginning varices during pregnancy	•	•	•	•		•		•
Prophylaxis of venous disorders for mobile patients								
CVI stage 1	•	•	•	•	•	•		•
After thrombophlebitis	•	•	•	•	•	•	•	•
Primary varices								
Secondary varices	•	•	•	•	•	•		•
Distinctive varices during pregnancy								
Vein stripping								
Sclerotherapy								
After removal of leg veins (e.g. bypass surgery heart)	•	•	•	•	•	•		•
CVI stage 2			•	•	•	•	•	•
Atrophie blanche					•		•	•
Phlebothrombosis					•		•	•
After healing of heavy venous ulcers					•		•	•
Thrombophlebitis			•	•	•	•	•	•
Severe secondary varices					•		•	•
CVI stage 3					•		•	•
Post-thrombotic syndrome					•		•	•

Classification regarding CEAP (Clinical Etiology Anatomy Pathophysiology)

Clinical classification	Juzo Naturally Sheer	Juzo Power Comfort, Cotton	Juzo Power Lite, Vibe	Juzo Power Wool	Juzo Basic	Juzo Soft	Juzo Dynamic	Juzo Move	Juzo Silver	Juzo Ulcer Pro	Juzo Compression Wrap
C ₁ Spider veins, reticular varices	•	•	•	•	•	•		•			
C ₂ Varices	•	•	•	•	•	•	•	•	•		•
C ₃ Edema		•	•	•	•	•	•	•	•		•
C _{4a} Pigmentation, purpura, eczema									•		•
C _{4b} Hypodermitis, lipodermatosclerosis, atrophie blanche							•		•		•
C ₅ Healed ulcus cruris venosum					•	•	•	•	•		•
C ₆ Active ulcus cruris venosum							•		•	•	•

Etiologic classification

Ec congenital

Ep primary

Es secondary (post-thrombotic)

Anatomic classification

s superficial veins

p perforator veins

d deep veins

Pathophysiologic classification

Pr reflux

Po obstruction

Pr,o reflux and obstruction

Prescription Recommendation for Lymphedema & Lipedema

Indication	Stage	Symptom	DECONGESTION PHASE	MAINTENANCE PHASE									
			Juzo Compression Wrap	Juzo Night	Juzo Move	Juzo Basic	Juzo Dynamic/Dynamic Silver	Juzo Soft	Juzo Sensation	Juzo Expert / Expert Cotton	Juzo Expert Silver	Juzo Strong	Juzo Strong Silver
Lymphedema	1	Spontaneously reversible, soft swelling, pressing on the skin leaves a pit, no skin changes	•	•	•	•	•	•	•	•	•		
	2	Spontaneously irreversible, onset of fibrosis, very little pitting, sometimes skin changes	•	•							•	•	•
	3	Advanced form, elephantiasis, fibrosis, chronic skin changes	•	•									•
Lipedema	1	Fine-nodule skin surface (orange peel skin), fatty swelling in the hip area			•	•	•	•	•	•	•		
	2	Coarse-nodule, uneven skin surface with large depressions									•	•	•
	3	Large, deformed skin flaps and lobules, with the tissue being firmer and harder	•								•	•	•

Scan to download the
Juzo Digital Catalog & Decision Tree App

